Prior to use, carefully read the instructions for use.

EN

PATTERN RESIN™ LS ACRYLIC RESIN FOR PATTERNS

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

- To make the pattern of:
 - tapered coping for telescopic crowns and other telescopic crowns.
 - inlay, palatal and lingual bar, connector and clasp.
 - Maryland bridge.
- Temporary sprill for soldering.
- In the mouth for core build up pattern.
- Construction of implant superstructures.

CONTRAINDICATIONS

In susceptible individuals, sensitisation to the product cannot be excluded. Use of the product should be discontinued if allergic reactions are observed.

WARNING
Contains methyl methacrylate. When used directly in the mouth for core build up particular care should be taken to avoid contact with the skin or oral mucosa. In case of accidental contact, immediately remove the offending material and flush the surrounding area liberally with water.

DIRECTIONS FOR USE

1. Powder and liquid preparation
Dispense adequate amounts of powder and liquid into each rubber cup (Fig. 1).

2. Building the coping (for secondary crown patterns of tapered crowns or telescopic crowns)
a) Moisten the tip of the brush with the liquid and pick up a small amount of powder (Fig. 2).

b) Deposit this mixture on the crown and progressively build up the secondary crown pattern by repeating the procedure (Fig. 3).
- In this case, no separating medium is required, but when working on a stone model, GC Acro Sep, GC Multi-Sep or Vaseline should be used as separate wax/vulcanizing agent.
- Clean the brush with liquid after use or during the work if necessary.

3. Checking the internal surface of the pattern
After the resin has set (3 min. at 23°C/73°F), remove the pattern carefully from the internal crown and check the margin and the internal surface (Fig. 4).

4. Fine adjustment

a) Return the pattern to the internal crown and make a uniform thickness (0.3-0.4 mm) by using a bur (Fig. 5).
b) Remove the pattern and check the thickness with the measuring device (Fig. 6).

5. Waxing veneer and Cut out
a) Perform the wax-up with inlay wax (Fig. 7). Note: There is no need to clean the surface of the pattern resin with a cleaning agent before waxing.
b) Cut out the veneer for the facing (Fig. 8).
c) Place the retention beads (Fig. 9).

6. Investing, Burning out and Casting
For a single crown, all procedures can be performed in the usual manner. For a larger pattern, hold at 250°C (482°F) for 1 hour before raising to the final burn out temperature (Fig. 10).

STORAGE

Recommended for optimal performance, store in a cool place away from direct sunlight.

NOTES

- Do not mix with other chemically cured resins.
- Do not return the residual powder and liquid to the bottle.
- In case of contact with skin wash off with soap and water.

PACKAGES:
1. Powder: 100g, 1 kg
2. Liquid: 100g (105mL), 250g (262mL)
3. 1-1 pkg.: 100g powder, 100g (105mL) liquid with accessories

CAUTION

- The liquid is highly FLAMMABLE. Avoid sources of ignition.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS.

Always familiarize yourself with the safety data sheets available at: <http://www.gceurope.com>

They can also be obtained from your supplier.

CLEANING AND DISINFECTING

MULTI-USE DELIVERY SYSTEMS: To avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use respect device and label for deterioration. Discard device if damaged. DO NOT IMMERGE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

Undesired effects - Reporting
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:

https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gcc.dental

In this way you will contribute to improve the safety of this product.

Last revised: 08/2019

https://ec.europa.eu/growth/sectors/medical-devices/contacts_en

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https://ec.europa.eu/growth/sectors/medical-devices/contacts_en

MANUFACTURED BY:
GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803, U.S.A.

DISTRIBUTED BY:
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-Ku, Tokyo 174-8585, Japan

EU: GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33, B-3001 Leuven, Belgium
TEL: +32 16 74 10 00

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