
roeko Parotisroll anatomic

roeko Parotisroll

Lingual

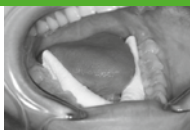Buccal

Labial

roeko
Parotisroll
anatomic

roeko
+ Parotisroll

Gebrauchsanweisung	DE	Instructions for use	EN	Mode d'emploi	FR	Istruzioni per l'uso	IT	Instrucciones de uso	ES	Instruções de uso	PT
--------------------	----	----------------------	----	---------------	----	----------------------	----	----------------------	----	-------------------	----

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Parotisroll and *Parotisroll anatomic* are dental rolls with a spring action inside. They temporarily reduce saliva flow, keep the mouth open and absorb moisture. The flattened ends of *Parotisroll anatomic* make it easy to apply, even if there is little space for working – both in the upper molar area as well as in the premolar area of the lower jaw.

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Tips for use Lingual

When used lingually *Parotisroll* is held in place by the spring action and the tongue is gently held down. *Parotisroll anatomic* is placed so that the flat surface exerts pressure on the inside of the dental arch. The flattened ends make *Parotisroll anatomic* easy to place in the narrow lingual area. There is less pressure on the area which causes gagging than with standard dental rolls. Dental treatment will be more pleasant and smooth both for the sensitive patients who tend to gag and the dental team.

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Buccal

When used buccally *Parotisroll* retracts the cheek and the mouth is kept open by the spring action. The roll exerts gentle but firm pressure on the Parotis roll thus reducing temporarily the saliva flow. *Parotisroll anatomic* is placed with the flat surface against the outside of the dental arch.

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Labial

When used labially *Parotisroll* retracts the lip and provides sufficient space for treatment in the cervical area. It is therefore ideal for use during esthetic dentistry. Also for orthodontics or when treating children it is of great help. *Parotisroll anatomic* is placed around the dental arch with the flat surface against the arch.

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Parotisroll anatomic in Spenderbox
10 mm 100 mm 50 Stck. REF 160 013

Parotimat – for hygienic storage and easy, single handed removal of *Parotisroll* and *Parotisroll anatomic*.
Parotimat REF 162 100

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Coltene/Whaledent GmbH + Co. KG